



HOW TO

NIV

ON THE



LTV

Step 1. Switch on.

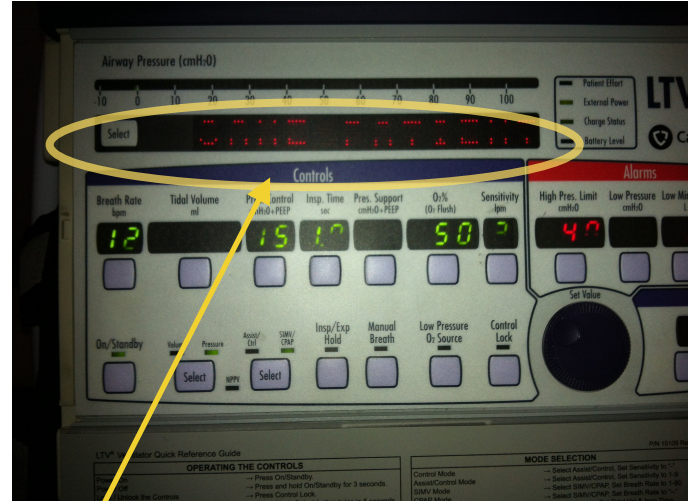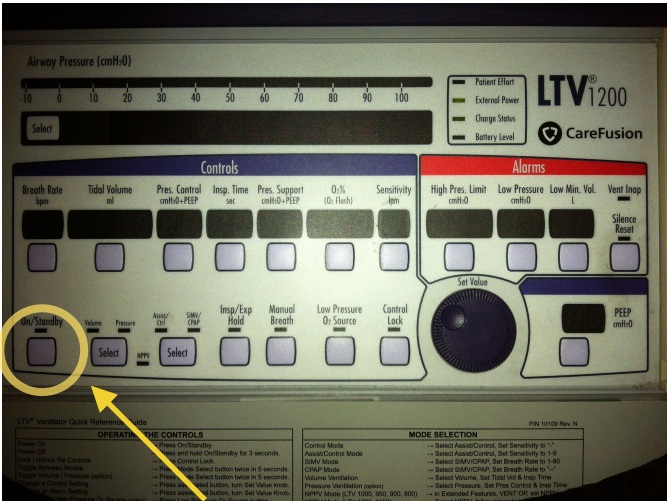



Press and hold "On/Standby" until it turns on and "same patient" appears in the screen.

Step 2. Select "New Patient".

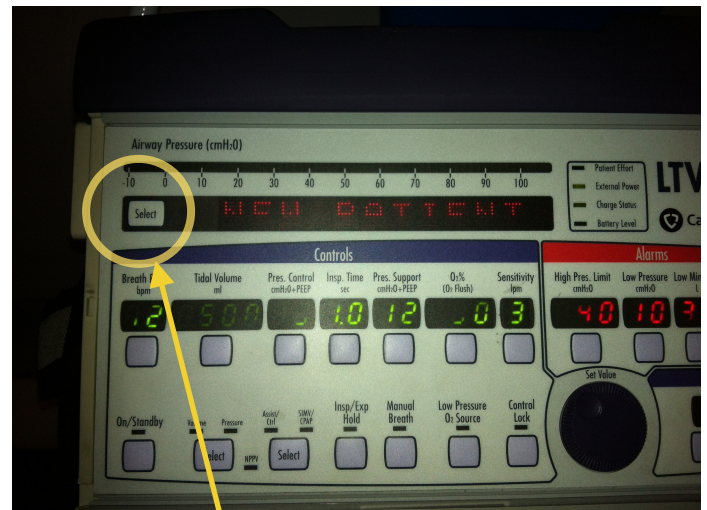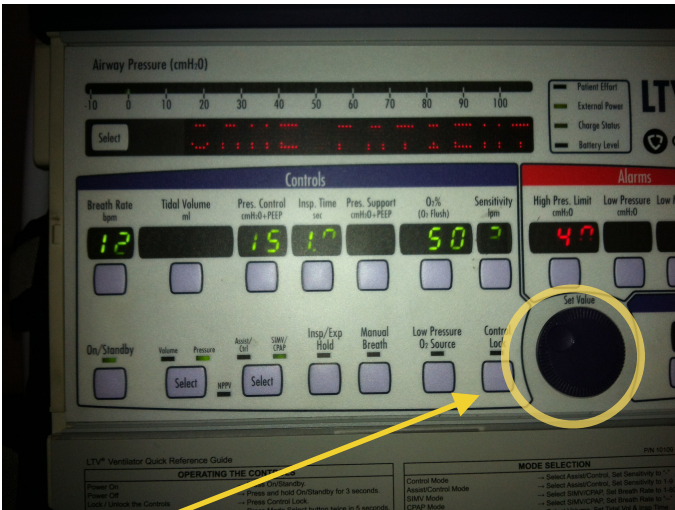


Turn the dial until "new patient" appears in the screen then press "Select".

Step 3. Select Adult Settings

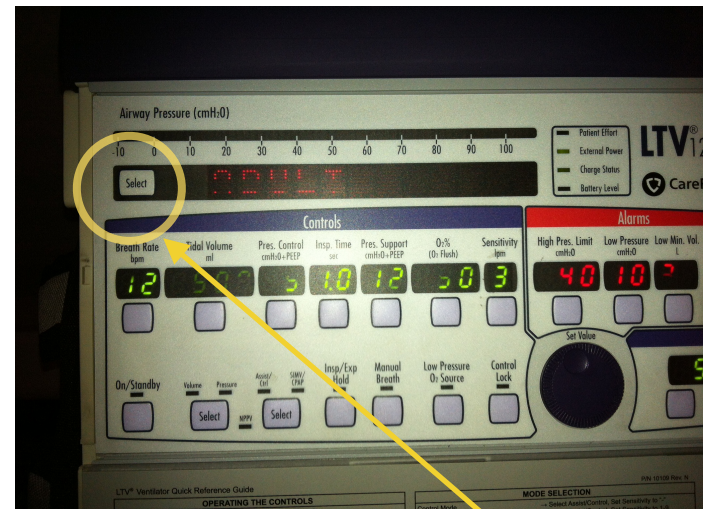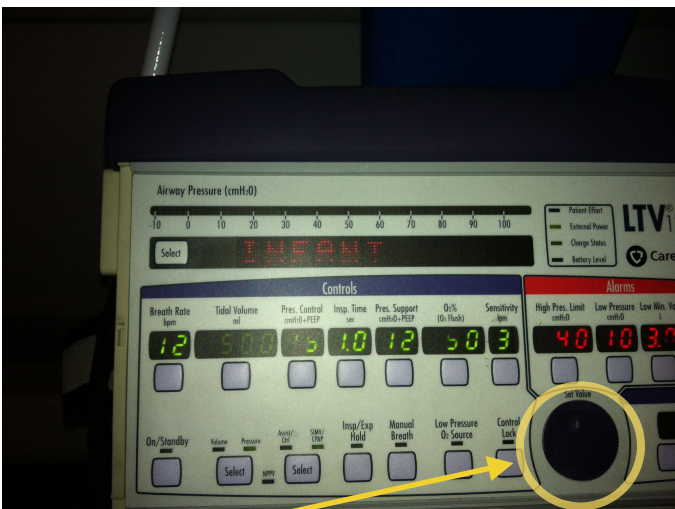

Turn the dial to change from "Infant" to "Paed" and then "Adult" on the screen then press "Select"

Step 4. Select NIV mode.

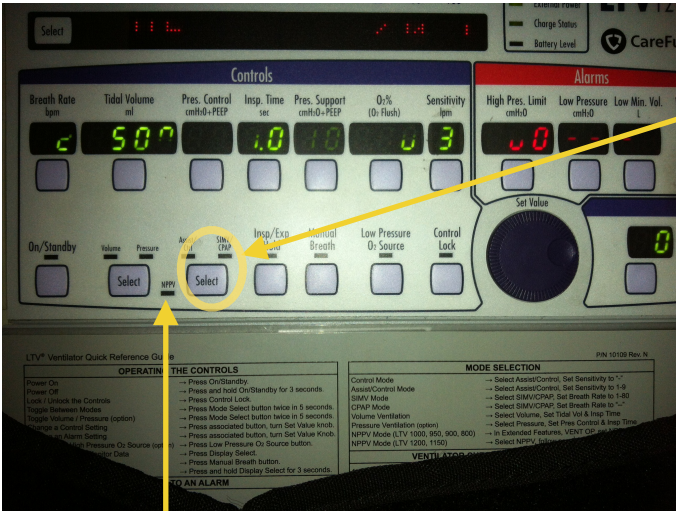

Press ventilator mode select button twice to change from Assist/Ctrl to SIMV/CPAP and then twice more to get to NIV settings.



The NIV light will flash.

NB. UNTIL THE IPAP AND EPAP SETTINGS HAVE BEEN CONFIRMED NIV IS STILL NOT OPERATIONAL.

STEP 5. Set IPAP values.

The Pressure Support display will be highlighted.


Turn the dial to select desired inspiratory pressure. (This will be the pressure above the selected EPAP/PEEP).

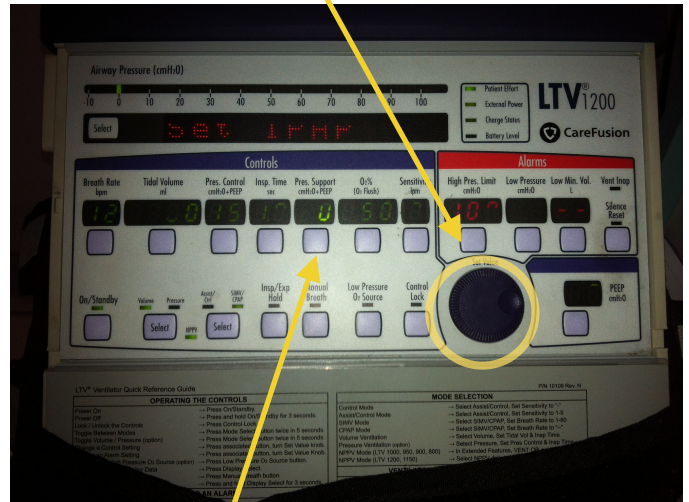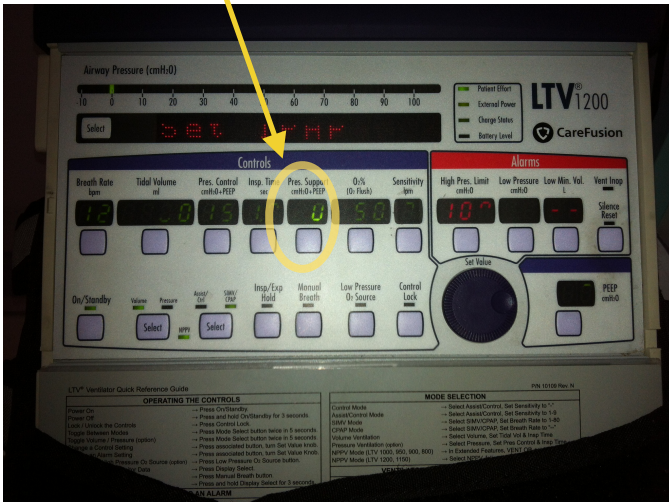

Press the "Pressure Support" to confirm.

Step 6. Set EPAP values.

The PEEP display will be highlighted. Turn the dial to select desired expiratory pressure.

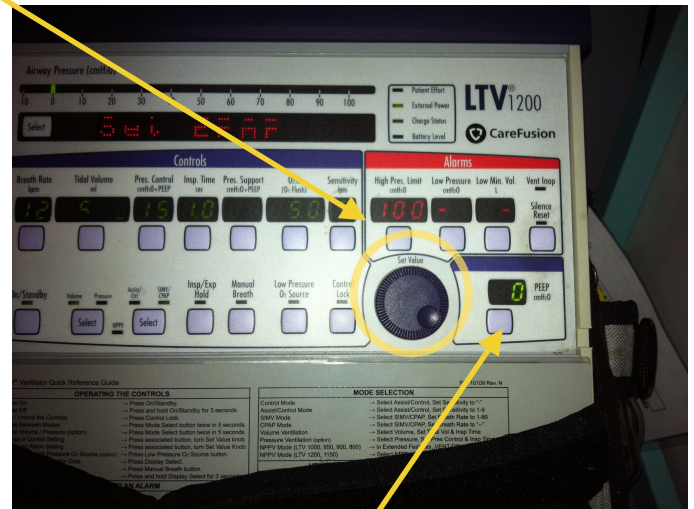


Press "PEEP" to confirm.

THE VENTILATOR IS NOW IN NIV MODE.

STEP 7. Attaching to patient.

Connect an HME filter to the ventilator tubing.

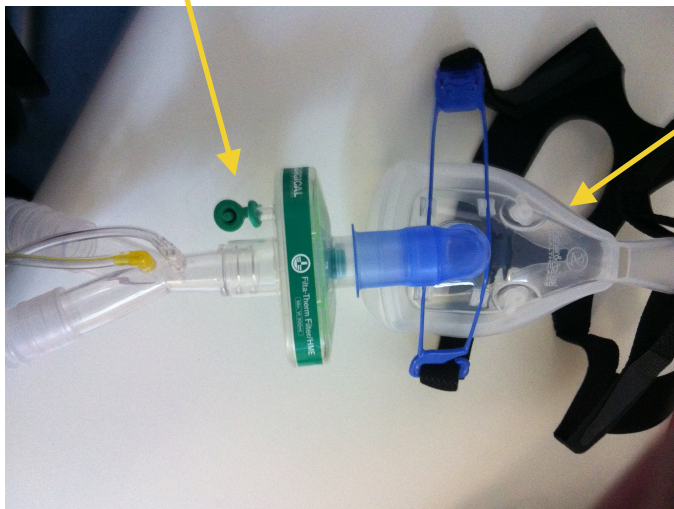


Apply a NIV Masks for the LTV (with blue connector) to the HME.

NOW APPLY THE MASK TO THE PATIENT AND SECURE IT WITH THE STRAPS.

Step 8. Adjusting Settings

IPAP. Press "Pressure Support", turn dial to select the desired pressure then press "Pressure Support" to confirm.


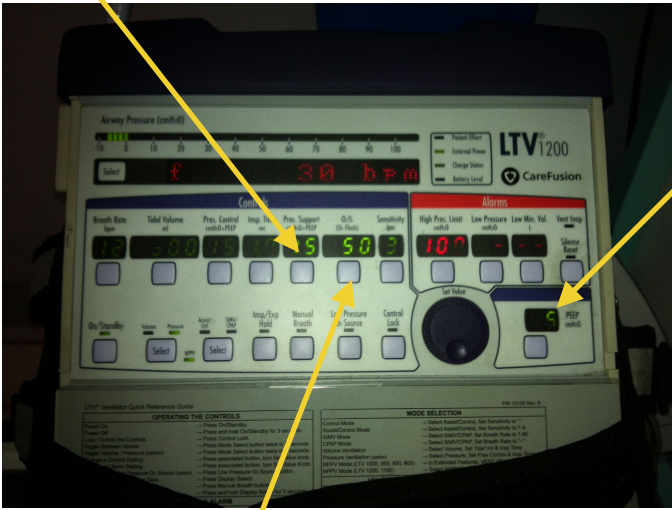


EPAP. Press "PEEP", turn the dial to select the desired pressure then press "PEEP" button again to confirm.

Oxygen. Press "O2%", turn the dial to select the desired FiO2 then press "O2%" again to confirm.

Switching Off

Press and hold "On/Standby" until the ventilator stops and the alarm sounds.

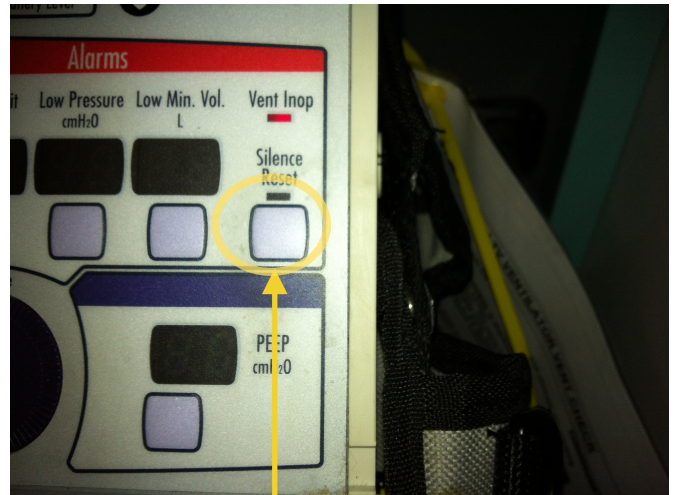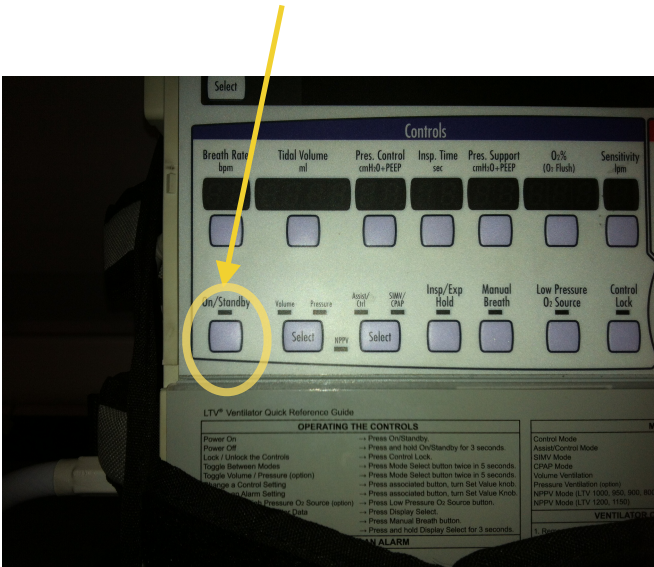

Press "Silence/Reset".